

Title Screen

Copy:

How to apply your FreeStyle Libre 2 sensor

The FreeStyle Libre 2 flash glucose monitoring system is indicated for measuring interstitial fluid glucose levels in people aged 4 years and older with diabetes mellitus. Always read and follow the label/insert.

VFX: Yellow half circle transitions in.

Copy:

Prepping your skin

VFX: Yellow half circle transitions off screen. Video footage of woman demonstrating WASH CLEAN DRY.

Copy:

WASH

Only use non-moisturizing, fragrance-free soap to wash your sensor application area, and then dry it

Copy:

CLEAN

Use an alcohol wipe (not alcohol gel) to remove any oily residue

Copy:

DRY

Allow your skin to air dry fully before applying the sensor. This is especially important if you plan to apply it after you shower or swim*.

*Sensor is water-resistant in up to 1 meter (3 feet) of water. Do not immerse longer than 30 minutes.

VFX: Video fades as type builds

Copy:

WASH CLEAN DRY BEFORE YOU APPLY

VFX: Video of woman applying FSL2 begins

Copy:

Confirm the code on the sensor pack and applicator match

Important: Sensor code on sensor pack and sensor applicator must match or glucose readings may be inaccurate.

Copy:

Peel the lid from the sensor pack

Important: Sensor code on sensor pack and sensor applicator must match or glucose readings may be inaccurate.

Copy:

Unscrew the cap from the sensor applicator

The Sensor Applicator contains a needle. Do NOT touch inside the Sensor Applicator or put it back into the Sensor Pack. Do NOT push down on the Sensor Applicator until placed over prepared site to prevent unintended results or injury.

Copy:

Place the applicator into the open sensor pack

The Sensor Applicator contains a needle. Do NOT touch inside the Sensor Applicator or put it back into the Sensor Pack. Do NOT push down on the Sensor Applicator until placed over prepared site to prevent unintended results or injury.

VFX: Arrows highlight alignment of marks

Copy:

Align the marks on the sensor applicator and pack

Copy:

Press down firmly on the applicator until it comes to a stop

Copy:

Lift the sensor applicator out of the sensor pack

Copy:

Place the applicator over the prepared area on the back of your upper arm

Copy:

Press firmly, listen for the click and hold for a few seconds

CAUTION: Do NOT use if the Sensor Pack or the Sensor Applicator seem to be damaged or already opened. Do NOT use if past expiration date.

Copy:

Slowly pull the applicator away

CAUTION: Do NOT use if the Sensor Pack or the Sensor Applicator seem to be damaged or already opened. Do NOT use if past expiration date.

Copy:

Run your finger along the sensor adhesive to secure it

Applying the sensor may cause bruising or bleeding. If there is bleeding that does not stop, remove the sensor, and apply a new one at a different site.

VFX: Yellow transition fills screen.

Copy:

It's that easy! Now your sensor is ready to go*.

* 60-minute warm-up required when applying the sensor.

VFX: FSL2 logo grows. Sonic mark animates.

Copy:

The FreeStyle Libre 2 flash glucose monitoring system is indicated for measuring interstitial fluid glucose levels in people aged 4 years and older with diabetes mellitus. Always read and follow the label/insert. —

The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott. —

© 2024 Abbott. ADC-90702 v1.0