

Mavericks How To - FSL2 PLUS

Video 5 - Using the Freestyle Libre 2 App to share glucose data with your doctor using Libreview

*This script applies to both MG and MMOL formats (only difference is glucose and ADC numbers)*

MG ADC- 90708

MMOL ADC-90708

<b>Audio</b>
<b>*MUSIC*</b>
<b>MOM VO:</b> One of the great things about...
<b>MOM VO:</b> ...my son's FreeStyle Libre 2 System...
<b>MOM VO:</b> ...is how much glucose information he has right at his fingertips...
<b>MOM VO:</b> ...updated every minute on his smart-phone.
<b>MOM VO:</b> And he has a LibreView account, so his glucose data is automatically sent to his doctor between visits.
<b>ANNOUNCER VO:</b> LibreView is a secure, cloud-based system...
<b>ANNOUNCER VO:</b> ...primarily used by your doctor, to manage patients on the FreeStyle Libre family of products.

**ANNOUNCER VO:**

Once you're connected, they receive your glucose information in the form of Clinical Reports...

**ANNOUNCER VO:**

...which they can use to better understand how to help you manage your diabetes.

**ANNOUNCER VO:**

Connecting with your doctor is easy.

**ANNOUNCER VO:**

The first step is to set up a LibreView account, if you don't already have one.

**ANNOUNCER VO:**

You can create an account right from the FreeStyle Libre 2 App.

**ANNOUNCER VO:**

Next, you'll need to establish a connection between your LibreView account ...

**ANNOUNCER VO:**

... and your doctor's LibreView account.

There are two ways to do this...

**ANNOUNCER VO:**

One is to share the email address you used to create your LibreView account with your doctor...

...and ask them to invite you to connect.

**ANNOUNCER VO:**

The other is to ask them for their LibreView practice I.D. and enter it into the FreeStyle Libre 2 App.

**ANNOUNCER VO:**

To connect with your doctor, go to the FreeStyle Libre 2 home screen on the menu.

**ANNOUNCER VO:**

Then tap CONNECTED APPS.

**ANNOUNCER VO:**

Then tap CONNECT or MANAGE in LibreView.

**ANNOUNCER VO:**

If your doctor used your email address to invite you, you will see a PENDING INVITATION.

**ANNOUNCER VO:**

If your doctor shared their practice I.D. with you, you will tap on CONNECT TO A PRACTICE and input it here. Follow the prompts to complete setup.

**ANNOUNCER VO:**

Your doctor will use your reports...

**ANNOUNCER VO:**

...including Time In Range, to help identify glucose patterns and trends...

**ANNOUNCER VO:**

...so they can make more informed treatment decisions regarding your diabetes management.

And they can use LibreView from any internet-connected device.

**MOM VO:**

It's great that Daniel can easily share...

...his glucose information with his doctor...

...right from his phone.

**MOM VO:**

And his doctor can view his glucose trends and patterns to help him manage his diabetes remotely.

**\*MUSIC\***