

Moving from the FreeStyle Libre System to the FreeStyle Libre 2 System



There is now a new version of the FreeStyle Libre System. It's called FreeStyle Libre 2.



There is a new type of sensor with FreeStyle Libre 2 System.

Moving from the old System to the new one is easy.



All you have to do is...

Talk to your doctor and confirm that you are changing to the FreeStyle Libre 2 sensor.



If you use a phone to scan your sensor

You can start scanning the new sensor straight away.



If you use a reader to scan your sensor

You need a new FreeStyle Libre 2 blue reader.



You can ask for one at:

www.FreeStyleLibre.co.uk/replacement

On this website, type in your current FreeStyle Libre reader **serial number**.



Each reader has some letters and numbers that help to identify it called a **serial number**, like this: JCMX000-K0000. Each serial number is different.

You can find your serial number in the grey square on the back of your reader.



You don't have to send your old FreeStyle Libre black reader back.

A new FreeStyle Libre 2 blue reader will be delivered directly to you.

What is the difference between the old and new Systems?

With the new FreeStyle Libre 2 System, there are:



- no finger prick tests - unless you feel different to what the reading and alarm are saying



- better and clearer pieces of information collected that will help you and your doctor manage your diabetes



- alarms which you can set to tell you if your sugar levels are too low or too high.



You can only set up an alarm on one of your devices at a time.



If you want the alarms on your reader, you need to start the FreeStyle Libre 2 sensor with the FreeStyle Libre 2 blue reader.



If you want your alarms on your phone, you need to start the FreeStyle Libre 2 sensor with the app on your phone.



You will need to scan your sensor every 8 hours to get a full picture of your sugar levels.



You also need to scan every time you get an alarm, to get your reading.



Your doctor will help you decide what to do.

For more information and training



If you need more information please call the Abbott Customer Careline on: **0800 170 1177**

Monday to Friday between 8:00am and 8:00pm, Saturday 9:00am and 5:00pm



Or you can go to our website: **www.FreeStyleDiabetes.co.uk**

The pictures in this document are only to help you understand it.

The people are not real patients. The sugar levels are not real patients' numbers.

FreeStyle **Libre 2** 

 **Abbott**
life. to the fullest.®

© 2021 Abbott. FreeStyle, Libre, and related brand marks are marks of Abbott.

ADC-36214 v1.0 04/2021.

Easy Read by **easy-read-online.co.uk**